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(EN) Instructions for use
(MD) EU Medical device
In accordance with DIN EN ISO 3107

Product description:

Omnitemp C, a eugenol-free cement with calcium hydroxide, is easy to process and does not influence the setting of resin-based filling and luting materials.

Omnitemp C contains calcium hydroxide to support tooth vitality.

Indications:

Omnitemp C is used as a temporary cement for the attachment and for sealing of small one-surface cavities.

Contraindications:

Omnitemp C contains modified colophony. Omnitemp C should therefore not be used in patients with a known hypersensitivity (allergy) to this ingredient.

Patient target group:

Omnitemp C is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

Omnitemp C should only be used by a professionally trained dental practitioner.

Use:

Due to technical reasons slight differences in the filling level between the syringe chambers can occur prior to the first use. Therefore remove syringe cap and squeeze the material out until the material is delivered evenly from both chambers. Place and lock the mixing tip with a 90° turn clockwise. The material is automatically mixed in the mixing tip.

All syringes come with a detachable piston. The piston can simply be pulled from the syringe and replaced without causing any material leakage. The plugs in the syringe chambers remain in place preventing cross contamination of the materials.

Application:

Apply a thin layer of Omnitemp C inside the temporary crown or bridge and affix it to the prepared tooth. Omnitemp C remains workable for about 1 min. Omnitemp C sets in the mouth within 4 - 6 min. Place the temporary into the mouth. Remove any excess material after 3 - 5 min.

Warnings, precautionary measures:

- In case of opened pulp or inferior dentine thickness, an adequate pulp protection is recommended.
- The sensitivity of cores can be reduced by use of desensitising preparations prior to temporary luting. Remove these preparations completely before definite luting (do not use preparations containing eugenol!). Extensive superficial and multisurfaced cavities as well as temporary longterm sealing work should be treated with a harder cement.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Supplementary indications:

Residues of Omnitemp C can be removed from crowns easily after they have been sprayed for a short time with a cooling spray.

Storage:

Store at 4°C-23°C. Do not use after the expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of Omnitemp C must be reported to the manufacturer and the responsible authority.

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MP Medical Product GmbH