**OMNI
DENT®**
We care. You smile.

Omnifill C Applikator

EU Medizinprodukt

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EU Medical device

GB Instructions for use

Applikator for the direct administration of glass ionomer cement restorative and luting materials in application capsules (AC). The fine aperture on the capsule in combination with the Omnifill C Applikator allows the material to be precisely applied to the cavity or the restoration for attachment.

Patient target group:


Omnifill C Applikator is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.


User:

Omnifill C Applikator should only be used by a professionally trained dental practitioner.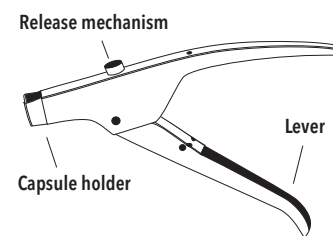


Application:


1. Make sure the applicator plunger is retracted/unlocked.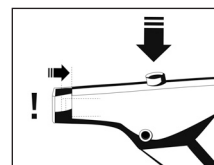


2. Following activation and completion of mixing, open the application capsule in accordance with the manufacturer's instructions.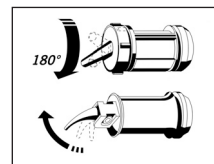

3. Insert the application capsule into the applicator, rotate in the desired direction and apply its content into the cavity or the restoration to be attached by slowly and evenly compressing the lever. Do not apply excessive pressure.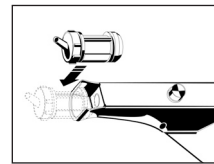

4. On completing the application, move the plunger back again by pressing on the release button and lift the application capsule out.

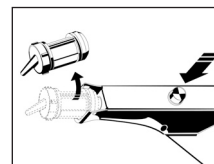5. Disinfect the applicator after each use. Conventional disinfectant wipes or solutions can be used for disinfection. The Omnifill C Applikator can be steam sterilised at up to a maximum of 134 °C (steam steriliser, follow manufacturer's instructions).

Warnings, precautionary measures:

- The Omnifill C Applikator application system enables you to quickly and hygienically apply the material directly.
- For detailed information on the different OMNIDENT products, please consult the individual instructions for use.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of Omnifill C Applikator must be reported to the manufacturer and the responsible authority.

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