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Produktbeschreibung

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Instructions for use

Product description

OmniCeram évolution 2.0 is a **visible light curing nanohybrid composite** for anterior and posterior restorations. It is based on BIS-GMA-resin and inorganic filler particles of 0.05 – 3 µm. The total filler load is 80 %, the total filler volume is 69 %. Radiopacity of OmniCeram évolution 2.0 is 200 % aluminium (radiopacity of enamel: ~200 % aluminium, of dentine: ~ 100 % aluminium). The composite material meets the requirements of **DIN EN ISO 4049**, type 1, class 2, group 1.

Indications/Intended use

Restorations of all cavity classes.

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of calcium hydroxide material.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Incompatibility with Other Materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the composite. Neither store the composite material in proximity of eugenol containing products, nor let the composite allow coming into contact with materials containing eugenol.

Application

Preparation of Tooth and Color Matching

Clean the tooth with a fluoride-free polishing paste (e.g. flour of pumice with water) prior to preparation and color matching. Ascertain the tooth shade while teeth are still moist and select the appropriate OmniCeram évolution 2.0 shade. OmniCeram évolution 2.0 is shaded according to VITA®-Shades. It is also available in special shades and bleach shades.

Cavity Preparation

After isolation (best with a rubber dam) prepare the cavity with minimal tooth reduction. For pulp protection areas close to the pulp should be covered with a thin layer of calcium hydroxide material. Condition and apply a bonding agent according to the manufacturer instructions. The total etch technique with Etching Gel and subsequent application of a light cure bonding agent (e.g. OmniCite 2.0) is recommended. Otherwise use a self etching bonding agent (e.g. OmniCite All'N'One) before application of the OmniCeram évolution 2.0.

Application and Curing of OmniCeram évolution 2.0

Apply OmniCeram évolution 2.0 in the selected shade. OmniCeram évolution 2.0 is easy to model. Place it by using a suitable instrument. Matrix strips may be used. In case of deep and extended cavities best results are obtained with a build-up of a base liner using a light cure resin modified cement or a light cure floatable composite (e.g. OmniCeram évolution flow 2.0), followed by the application of OmniCeram évolution 2.0. In case of small cavities OmniCeram évolution 2.0 can be applied directly.

For an optimum result apply OmniCeram évolution 2.0 in **layers of max. 2 mm**. Light cure each increment separately.

By using a polymerization unit (wavelength range 400-500 NM) **with a light intensity of at least 1000mW/cm²**, cure each increment as follows:
Lighter shades (e.g. A1, A2, A3, B1, B2, Bleach) **20 seconds**
Darker shades (e.g. A3.5, C3, grey) **30 seconds**

Hold the light emission window as close as possible to the filling material.

Finishing

Depending on the shape of the area contour with finishing diamonds, flexible grinding discs or carbide burs. Polish to high gloss with polishing discs or polishing brushes.

Storage

Do not store above 25°C (77°F). Protect from direct sunlight. Do not use after expiry date. Close syringe immediately after use to avoid exposure to light.

Additional Notes

- PL-Tips are for single use only due to hygienic reasons.
- The ambient light of the dental lamp may start polymerisation of the composite.
- Do not use any resin to adjust viscosity of composite restorative material.
- Avoid contact with skin, mucous membrane and eyes.
- Unpolymerized OmniCeram évolution 2.0 may have an irritant effect and can lead to sensitization against methacrylates.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- VITA® is a registered trade mark of the VITA-Zahnfabrik, Bad Säckingen, Germany.
- Keep away from children!

Composition

Dimethacrylates, dental glass, silicon dioxide, photo initiators.

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>).

Warranty

SanaPro Dental GmbH warrants this product will be free from defects in material and manufacture. SanaPro Dental GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and SanaPro Dental GmbH's sole obligation shall be repair or replacement of the SanaPro Dental GmbH product.

Limitation of Liability

Except where prohibited by law, SanaPro Dental GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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