# Metal Primer

Art.-Nr. 200942

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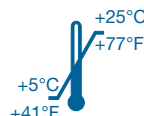
## Metal Primer

Art.-Nr. 200942

### dental-primer for bonding resin to metals and zircon

#### Range of applications

Bonding agent which promotes the adhesive strength of bonds from resins and composites to metals and zirconia.

#### Instructions for use

Bonding of acrylic and composite resins to metals (for example metal prostheses) and zirconia.

#### Application (general)

1. Sandblast metal/zirconia surfaces with aluminium oxide and blow clean with air gun. Avoid contamination with oil, separator or moisture. Do not clean surfaces with phosphoric acid.
2. Dispense 2 – 3 drops of Silanisation Liquid Metal Primer into a clean dispensing dish. Use a brush (must not lose hairs) to apply a thin layer to the bonding surface, avoid pooling when treating crowns. Allow to dry. To further strengthen the bond, immediately thereafter a second layer may be applied.
3. Immediately after treatment apply a resin material to the treated surface according to the instructions of the manufacturer.

#### Recommended applications (detailed)

1. Resin cement to metal for metal inlays, crowns and bridges: Prepare metal surface as above and bond restoration with resin cement.
2. Opaque resin to metal for composite resin veneers and crowns: Prepare surface as above and apply opaque in usual manner. Cure as recommended by the manufacturer.
3. Denture base to magnetic attachment: Prepare surface as above and bond attachment in usual manner using self curing acrylic resin.
4. Denture base resin to metal base, clasp or attachment:
  - a) Prepare bonding surface with alumina sandblast and blow clean with air gun.
  - b) Form a wax denture.
  - c) After flushing out the wax, clean the bonding surface using a steam cleaner. Remove wax completely using solvent.
  - d) Apply Silanisation Liquid Metal Primer.
  - e) Pack and cure denture base in usual manner.
5. In the mouth repair of damaged crown and bridge veneers:
  - a) Place rubber dam.
  - b) Clean metal surface with a carbide bur and prepare as above. Be sure to use rubber dam and avoid direct contact with oral tissue. In case of contact, immediately flush with water and seek medical attention.

6. Repair of denture with metal base, clasp or attachment:
  - a) Roughen metal bonding surface using carborundum point.
  - b) Prepare bonding surface as above.
  - c) Build up using self cure resin in the usual manner.
7. Bonding resin to metal orthodontic appliances:
  - a) Prepare bonding surfaces as above.
  - b) Apply orthodontic resin to the appliance in the usual manner.

#### Indications/precautions

1. For use only by dental professionals for the applications recommended.
2. Avoid contact with skin and mucous membrane.
3. Not to be used by operators, or on patients, with a history of allergy to methacrylates. In case of allergic response, seek medical attention.
4. In case of contact with skin immediately flush with water and soap.
5. In case of contact with eyes immediately flush with water and seek medical attention.
6. Flammable. Avoid high temperatures, direct sunlight and sources of ignition. Use in a well ventilated area.
7. Wear protective gloves.
8. Volatile. Seal container immediately after use.

#### Safety Measures

- H225 Highly flammable liquid and vapour.
- H317 May cause an allergic skin reaction.
- P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
- P233 Keep container tightly closed.
- P302+352 IF ON SKIN: Wash with plenty of water and soap.

#### Advice for storage and usage

- Store in a cool place (5 – 25 °C/41 °F – 77 °F) out of direct sunlight.
- Shelf-Life: 2 years from date of manufacture

#### Date of expiration: see packaging

#### Delivery forms

- Omnident Metal Primer
- 1 x 5 ml bottle Silanisation Liquid