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Omni Profi Varnish Gel is a fluoride containing one component varnish with Xylitol. For desensitizing of teeth it is applied on dentin and enamel surface. Omni Profi Varnish Gel contains in addition to easy soluble fluorides Calciumfluoride and Fluorapatite. Fluorapatite particles are known to enhance remineralization of tooth material. Omni Profi Varnish Gel has a pleasant sweetish taste.

Omni Profi Varnish Gel adheres to wet and dry surfaces and can be applied easily and time saving.

Omni Profi Varnish Gel contains 22600 ppm (2,26 %) fluoride.

Omni Profi Varnish Gel is designed for the following indications:

- protection and fluoridation of tooth surface
- treatment of hypersensitive teeth
- treatment of exposed dental necks
- sealing of teeth during and/or after orthodontic procedures
- sealing of tooth surface after bleaching
- Omni Profi Varnish Gel is for adults and for children over 3 years old

Application of Omni Profi Varnish Gel

Recommended dosage:

For treatment with Omni Profi Varnish Gel the following dosage is recommended for a complete dentition:

Teeth of 1. dentition: approx. 0.2 ml Omni Profi Varnish Gel

Permanent teeth: approx. 0.4 ml Omni Profi Varnish Gel

From experience, the required amount for a complete set of teeth ranges between 0.2 and 0.4 ml Omni Profi Varnish Gel.

A strand length of 1 cm on the mixing pad is equivalent to approx. 0.1 ml Omni Profi Varnish Gel.

For children from 3 to 6 years ensure that not more than 0.2 ml Omni Profi Varnish Gel is applied.

Omni Profi Varnish Gel should be applied as a thin film on the dental hard tissue. A thick film give no therapeutic advantages but peels off more easily.

Clean the teeth thoroughly from tartar and remove excess moisture and saliva from the areas to be treated. Place Omni Profi Varnish Gel on the mixing pad and apply it with the supplied brush on the teeth. The entire surface should be covered with an even thin film. Moisturizing by natural saliva ensures a proper setting of Omni Profi Varnish Gel.

Note:

During the first 4 hours after application of Omni Profi Varnish Gel avoid solids, alcohol, tooth cleaning with brush and floss.

Close tube immediately after use.

Remarks

- Omni Profi Varnish Gel contains natural resins.
- In case of known hypersensitivities (allergic reactions) to natural resins, Omni Profi Varnish Gel should not be used.
- Natural resins may inhibit the polymerization and adhesion of composites.
- To prevent the cap to stick, wipe-off any contaminating residues on the screw of the tube with an ethanol-containing cloth.
- Omni Profi Varnish Gel is contraindicated in patients with ulcerative gingivitis or stomatitis. Omni Profi Varnish Gel is not intended for systemic treatments.
- In case of protecting teeth with Omni Profi Varnish Gel during application of orthodontic multi-band technique, the brackets are placed on the teeth before the application of Omni Profi Varnish Gel.
- In case of hypersensitivity reactions, remove Omni Profi Varnish Gel with a soft tooth brush and floss and rinse with warm water.
- On the day of treatment use no other fluoride preparations. Stop treatment with fluoride tablets for several days.

- Do not swallow Omni Profi Varnish Gel.
- In case of swallowing large amounts of Omni Profi Varnish Gel consult a doctor immediately.
- Do not use Omni Profi Varnish Gel for children under 3 years.

Storage:

Do not store above 25 °C (77 °F)! Do not use after expiration date!

Omni Profi Varnish Gel is available in tubes with 10 ml and in blister with 0.3 ml content.

For hygienic reasons Omni Profi Varnish Gel in blister is for single use and for one patient only.

Warranty

SanaPro Dental GmbH warrants this product will be free from defects in material and manufacture. SanaPro Dental GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and SanaPro Dental GmbH's sole obligation shall be repair or replacement of the SanaPro Dental GmbH product.

Limitation of Liability

Except where prohibited by law, SanaPro Dental GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

**Keep away from children!
For dental use only!**

 **Manufacturer:**
SanaPro Dental GmbH
D-27570 Bremerhaven
Germany

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